Intended use

A transport system for bacteriological samples.

Principle of the method

The substrate in the Transpocult tube is a modification of Stuart medium. The nonnutritive, semisolid substrate preserves the microbes during transport from the sampling site to a laboratory, by preventing their dehydration, oxidation and growth. Oxidation of the medium is prevented by reducing components¹.

Contents

- 20 Transpocult transport medium tubes
- 20 sterile Dacron swabs
- 20 patient labels
- Instructions for use

Material required but not included:

Laboratory equipment for microbial culture.

Typical formulation

Agar	6,0 g
Sodium chloride	3,0 g
Sodium glycerophosphate	14,0 g
Sodium sulphite	1,0 g
Calcium chloride	0,13 g
Cysteine hydrochloride	0,03 g
Thioglycolic acid	0,5 ml

Warnings and precautions

- Transpocult is for IN VITRO DIAGNOSTIC USE only.
- Do not use the product beyond the expiry date.
- Do not use the Transpocult tube if you detect evidence of bacterial or fungal growth.

Storage

Transpocult tubes are stored at room temperature (18...25°C). The expiry date is marked on the box.

Sample collection

1. Remove the stopper.
2. Take the swab from the protective envelope and collect the sample.
3. Insert the swab into the medium and close the tube with the stopper.
4. Fill in the label and attach it to the tube.

Expected values

The transport medium is expected to maintain the bacteriological sample unchanged, the sample is expected to reflect the original situation, and the number of microbes is not expected to increase or decrease during transportation prior to culturing².

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Limitations of the procedure

If a bacterial sample cannot be transported to the laboratory for culturing within one working day, it should be chilled (2...8°C). Especially *N. gonorrhoeae*, *Haemophilus influenzae*, *Campylobacter jejuni* and other demanding microbes survive storage better refrigerated than at room temperature³.

Quality control

Quality control tests are performed on each lot of Transpocult at the time of manufacture. Should the user wish to perform his own quality control, the following procedure is recommended:

N. gonorrhoeae ATCC 43069

1. Prepare two separate tubes containing 0.1 ml of a 10⁵⁻⁶ CFU/ml *N. gonorrhoeae* suspension in sterile 0.9 % NaCl.
2. Absorb each suspension into a sterile Dacron swab.
 - Swab no. 1**
 - a. Insert the swab into the Transpocult gel. Recap the tube.
 - b. Place the Transpocult tube in a refrigerator for 24 h.
 - c. Take the swab from the Transpocult tube and use it to inoculate a Columbia blood agar plate.
 - d. Incubate the inoculated plate at 36 ± 2°C for 48 h.
 - Swab no. 2**
 - a. To confirm the presence of living *N. gonorrhoeae* cells in the 0.9 % NaCl suspension, use swab no. 2 to directly inoculate a Columbia blood agar plate.
 - b. Incubate the inoculated plate at 36 ± 2°C for 48 h.
3. After incubation, verify the presence of *N. gonorrhoeae* colonies on the two plates with oxidase reagent.

Disposal

Inoculated Transpocult tubes may contain highly contagious material. Inoculated tubes should be disposed of according to existing local regulations.

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Orion Diagnostica Oy